

# Module attachment and removal

## Alaris® System

### Module attachment

1. Inspect IUI connectors on each Alaris® PC unit and on each module prior to every use. *(Figure 1)*
2. Position the module at a 45° angle to the Alaris® PC unit or a pre-attached module.
3. Align the male to female interconnecting components.
4. Angle the module down until the latch snaps into place. *(Figure 2)*



Figure 1. IUI connector



Figure 2. Module attachment

### Module removal

1. Ensure module is powered **OFF** prior to removal. Push in on the module release button and angle the module away from the Alaris® PC unit or module to disengage the interconnecting components. *(Figure 3)*



Figure 3. Module removal

For more information please contact Technical Support at,

U.S.: 800.854.7128, ext. 6003

Canada: Eastern: 800.908.9918

Western: 800.908.9919

©2010 CareFusion Corporation or one of its subsidiaries. All rights reserved. Alaris® is a registered trademark of CareFusion Corporation or one of its subsidiaries. (IF1727/PDF)

CareFusion  
San Diego, CA

[carefusion.com](http://carefusion.com)

