



Delay Start Options with Delay Start Callback

Alaris® Pump module and Alaris Syringe module

What is the Delay Option feature?

The Delay Start Option allows the user to schedule/program an infusion to be **Delayed for** up to 120 minutes or **Delayed until** a specific time up to 23 hours 59 minutes. A Delay Start Callback also allows the user to select an audiovisual callback alert.

There are three types of callbacks:

Before: *Callback* can be scheduled to occur when the delay is completed and infusion needs to be initiated.




After: *Callback* can be scheduled to occur when the delayed infusion is complete.

Before and after: *Callback* can be scheduled to occur when delay is completed and infusion needs to be initiated and when delayed infusion is complete.

Note: With a delayed start it is assumed that another infusion would be running to keep the IV line patent until the delayed infusion begins.

How to program a Delay Start with a Callback:

1. Select **Delay Options** soft key on the Alaris® PC unit.
2. Select option for type of delay and enter in desired value. (Figure 1)
3. Prior to pressing **Confirm** to initiate the delay option, press the **Callback** soft key. (Figure 2)
4. Select type of callback desired.
5. To initiate the delay and the callback, press **Confirm**. (Figure 3)

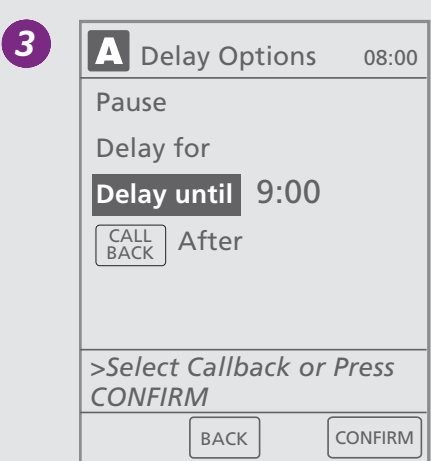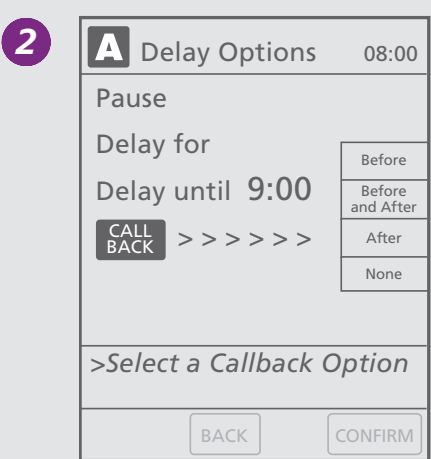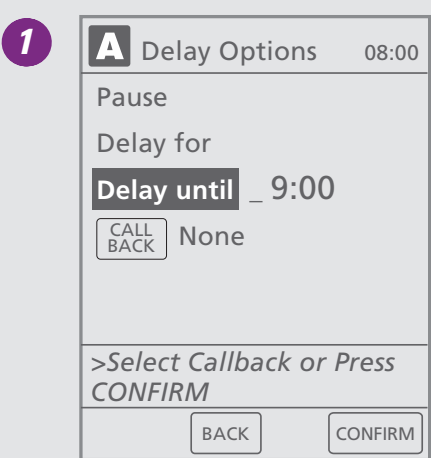

Note: There is no reversion to KVO at the end of an infusion when the Delay option is used, and the Alaris System will not display an audio or visual alert when the delayed infusion is completed unless a Callback option has been programmed by the user.

For product support, please contact Customer Advocacy at 888.812.3266 or email customerfeedback@carefusion.com.
For technical support, please contact Instrument Technical Support at 888.812.3229.
For product orders, please contact Customer Order Management at 800.482.4822.

CareFusion
San Diego, CA

carefusion.com

