

Important Notice:

Alaris Pump

To ensure the Alaris System remains in good operating condition both regular and preventative inspections are required. Inspect the pump prior to use to ensure the clear membrane that covers the upper pressure sensor and extends to the lower pressure sensor is **intact**. If the membrane is not intact, **do not use**, remove the pump from service and notify Biomedical Engineering.

Clear Membrane
Inspect closely



Upper Pressure Sensor
Upper Occluder
Upper Pumping Finger
Lower Occluder
Lower Pumping Finger
Lower Pressure Sensor