

Alaris Pump Library Upgrade/Upload Instructions:

1. Turn on the pump and look for the “NEW PATIENT?” message. Select “YES”.
2. Next screen will display the current library version date at the top of the screen and displays the Profile or asks you to choose a Profile. If the library version is **LMC 07 31 2013**, the version is current and you need to do nothing. If the date is NOT current, confirm the Profile or enter one and leave the unit on.
3. Watch the signal indicator at the bottom left corner of the pump (under the AC power indicator). After it blinks several times (receiving the download), around 30 seconds to one minute, it should stop blinking rapidly and remain on.



4. Then power the unit OFF and back ON. Follow steps 1 and 2 again. Then new library version should be displayed above the Profile. If not, repeat the procedure.

Please remember to update **ALL** the pumps on your unit that are **NOT** currently in use.

Do **NOT** update a pump while in use on a patient! This will occur the next time the pump is used on a “NEW PATIENT”.